

↑ Pull here ↑



Midstream

Universal urine collection kit (30 ml)
PE 40 Standard


This pack contains:

- 1. Genital wipe
- 2. Bottle (30 ml)
- 3. Peezy

Peezy works best with a full bladder

How to use Peezy in 5 easy steps - read before use


1 Use genital wipe

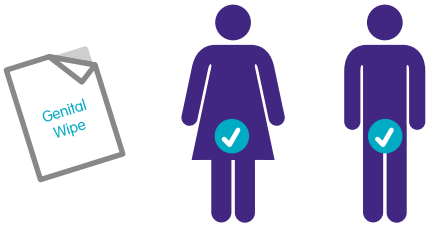

Genital Wipe

2 Fit bottle


- Remove lid.
- Screw bottle on to Peezy.

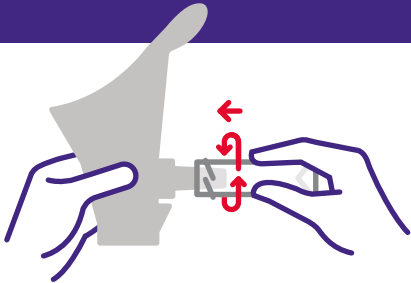

3 Pee


Female

Sit well back



Male

Hold over toilet or urinal




- Hold Peezy as shown and pee.
- Pee also runs into the toilet.
- **When finished count to 10** - this lets Peezy drain.


4 Remove bottle

- Hold bottle upright.
- Unscrew bottle.
- Replace lid.


Put Peezy in bin provided

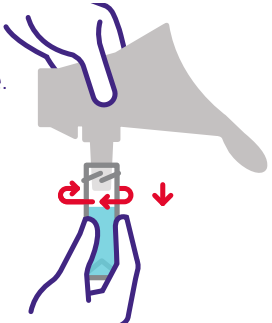

5 Give bottle to nurse

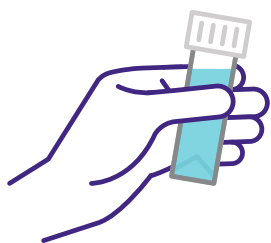

i How do I help someone use Peezy?

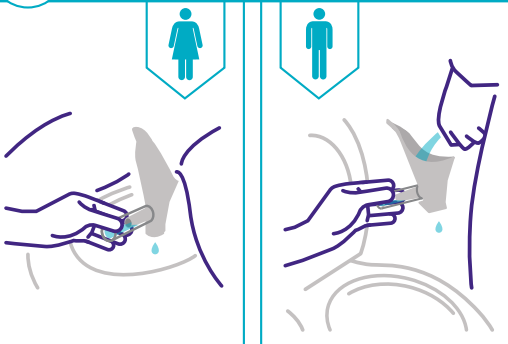

Do not use Peezy if the pack is open or damaged. Throw away and ask your nurse for a new pack.